

## Tutorials For You

Ready for action.

We know time counts. This is why Guardant Health wants to fit into your day-to-day operations.

Whether it is your first Guardant Health sample or you are a process champion already - here some tutorials for you.



### Draw the blood



Carefully fill & sign the Test Requisition Form



Use the Guardant Health provided kits only



Draw blood by venipuncture



Fill all tubes that come with the kit



Mix tubes gently 8-10 times



Label the tube



Put tubes back into the kit

### Ship the sample



Book a FedEx collection:

[HERE](#)



Book a FedEx collection:

- Cooling brick
- Foam tray with blood tubes inside the biohazard bag
- Put Test Requisition Form inside the biohazard bag window
- Cooling brick  
Do not freeze or refrigerate!



Close blood collection kit



Put kit inside the FedEx shipping bag & seal



Have the FedEx bag & shipping documents on hand



Handover to FedEx courier



We are on hand to answer any questions you may have and always aim to reply within one business day. Please call or email our expert Client Service team.

+44 808 189 5057  
[clientserviceeurope@guardanthealth.com](mailto:clientserviceeurope@guardanthealth.com)